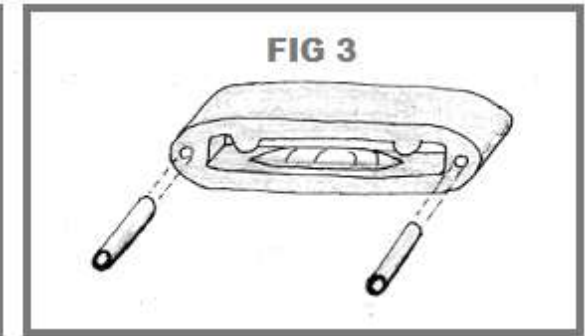
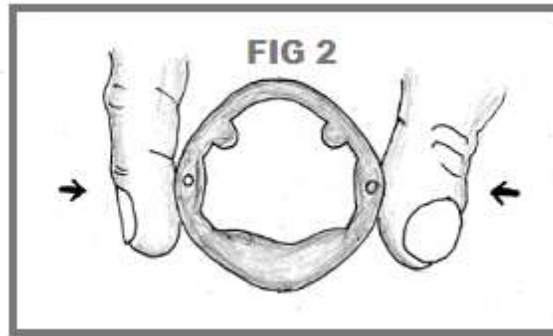
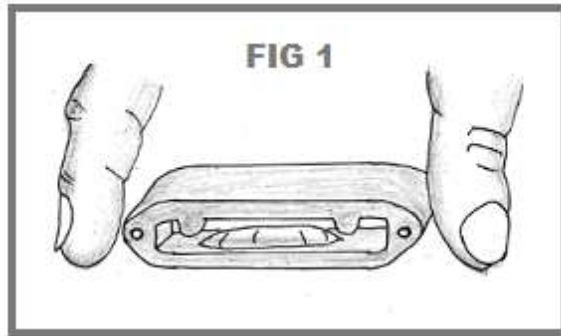
**Uriclak®**

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## ENGLISH

### PRESENTATION AND DESCRIPTION

URICLAK® is a transformable external controller for male urinary incontinence easy to use.

When pressed with fingers inwardly, URICLAK® becomes a circular ring than allow urinate without removing it, thus avoiding splashing.

URICLAK® is a medical device class I presented in a single size 91 mm long by 20 mm wide by 18 mm high, soft medical material coated SEBS USP Class VI certified and four pairs of cylinders with pressure regulators.

### INDICATION AND PRESCRIPTION

URICLAK® is indicated for male incontinence of stress, urge, mixed or dripping after urination.

It is contraindicated for patients with: functional incontinence due with brain damage, severe mental disabilities, or spinal cord injury, or overflow incontinence due to internal obstruction (with difficulty urinating).

URICLAK® is for adults only.

Do not need a prescription but you should tell your urologist is using URICLAK®.

## INSTRUCTIONS

URICLAK® is taken with the thumb and forefinger on the ends, with the elongated bulging towards and the two small bulges upward (FIG. 1).

Then you press Uriclak® with your fingers laterally inward (FIG. 2), becoming circular, and placed around the penis centering it between the two bumps.

**- IMPORTANT NOTICE: To prolong the life of the device, cleaning with a cotton swab slightly moistened with alcohol and avoiding contact with water.**

Each day before use, it is recommended to grease the elongated bulging with moisturizing milk to avoid possible irritation and pain in the skin.

If, despite this, the pressure on the urethra is too, **enter cylinders tip moistened with moisturizing milk** to facilitate its penetration into the holes as shown in FIG 3. **Always start with the smallest diameter.** A larger diameter is lower pressure.

We suggest urinate every one to two hours and so prevent the bladder is completely full,

If there are hairs on the skin of the penis, where the device is placed, it can remove them to avoid hitches.

In the case of light urine leakage despite having URICLAK®, you should use a compress level 1 male.

## PRECAUTIONS

**- Do not use in case the soft coating causes deterioration.**  
**- Do not use if inner metal structure breaks.**  
**- If any discomfort occurs, the device must be changed places (above or below).**

## POSSIBLE SIDE EFFECTS

URICLAK® usually causes no adverse effect. If you notice redness and can not be solved by changing pressure regulators or smearing moisturizing milk, discontinue use and consult an urologist.

## LEGAL NOTICE

URICLAK® is a patented and manufactured in accordance with Royal Decree 1591/2009 of Medical, European Directive

93/42/EEC and ISO 13485. URICLAK® is a registered trademark. All rights reserved

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