

# Maxhim<sup>®</sup>

Intimate Essentials for Men



**ERECTION CONSTRUCTION RING SET**

## THANK YOU FOR CHOOSING MAXHIM®

Please read this information carefully before using the Maxhim® Erection Constriction Ring Set. This device may cause injury if not used correctly.



The Maxhim® Erection Constriction Ring Set is a medical device used by men with light erectile dysfunction.

The Maxhim® Erection Constriction Ring Set is intended to improve the quality of erections.

Some men cannot maintain an erection because the veins of the penis stay open and prevent enough blood from being stored in the penis. This may be due to an existing health condition or as a result of prescription medication. The Maxhim® Erection Constriction Ring Set keeps the blood in the penis for longer and reduces blood flow back to the body.

### HOW TO USE

1. Stretch your chosen Maxhim® Erection Constriction Ring prior to arousal.
2. Place your testicles through the ring, followed your flaccid penis. The Maxhim® Erection Constriction Ring should be placed on the penis, between the testicles and the body.
3. As your arousal grows, the Maxhim® Erection Constriction Ring reduces the blood flow back to the body.
4. To loosen the tension, gently pull on the ring and slide off the penis, followed by the testicles.

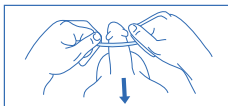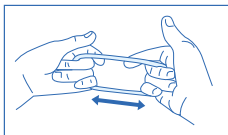

### WHICH SIZE RING TO CHOOSE

The right ring size helps you to achieve the best results.

If this is your first time using the Maxhim® Erection Constriction Ring Set, start by using the largest ring size.

If you find the largest ring size doesn't meet your needs, slowly work your way down in size until you find a comfortable and successful fit.

## **TAKING CARE OF YOUR MAXHIM® ERECTION CONSTRICTION RING SET**

Your Maxhim® Erection Constriction Ring Set is reusable. Clean the ring after use with warm water and mild soap. Keep and store in a dry place.

Replace your Maxhim® Erection Constriction Ring Set if the material becomes brittle or no longer has the tension to sustain your erection.

If using a lubricant, please use a water-based lubricant. Mineral-oil based lubricants may decrease the lifespan and flexibility of the Maxhim® Erection Constriction Ring Set.

## **WARNINGS AND PRECAUTIONS**

The Maxhim® Erection Constriction Ring Set is made of natural latex rubber which may cause latex allergic reactions.

- Do not use the Maxhim® Erection Constriction Ring Set for more than 30 minutes. Prolonged use may cause permanent injury to the penis.
- If a problem occurs, contact your doctor. Stop using the Maxhim® Erection Constriction Ring Set if problems persist.
- Do not use the Maxhim® Erection Constriction Ring Set if you cannot sense pain in the penis.
- Do not use the Maxhim® Erection Constriction Ring Set if you do not have adequate use of your hands to easily remove the device.
- Do not use alcohol or sedatives when using the Maxhim® Erection Constriction Ring Set.
- Do not use the Maxhim® Erection Constriction Ring Set as a method of contraception.

## **CAN WE HELP?**

If you experience issues or have any questions at all, please do not hesitate to contact our expert customer services team.

**Telephone: 01482 496931 / Email: [sales@stressnomore.co.uk](mailto:sales@stressnomore.co.uk)**  
(Mon to Fri: 8am – 6pm & Sat: 9am – 3:30pm GMT)

## DIRECTIONS FOR USE:

Comfortably maintain a strong erection for longer with the Maxhim® Erection Constriction Ring Set. Made of latex rubber, simply place your chosen erection ring around the penis and testicles to reduce blood flow out of the penis. See the full guide inside for more information.

Check out the full Maxhim® range:

[www.stressnomore.co.uk](http://www.stressnomore.co.uk)

**Maxhim**®  
Intimate Essentials for Men



Manufactured for  
Savantini Ltd  
Savantini House,  
Foster Street, HU8 8BT

IFU13226\_0819